Montage- und Gebrauchsanleitung  
Assembly and operation manual  
Instrucciones de uso y montaje  
Instructions de montage et d'utilisation  
Istruzioni di montaggio e d'uso  
Montage- en gebruiksaanwijzing

.bock<sup>®</sup> ///

Seitensicherung System SR2  
Side rail system SR2  
Barandillas SR2  
Système de barrières latérales SR2  
Sistema di sponde SR2  
Zijhekken systeem SR2

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für

- domiflex 1
- domiflex niedrig
- domiflex niedrig classic
- domiflex 2
- domiflex 2 wash
- domiflex 3
- domiflex 3 | 185
- combiflex
- belluno
- practico economic
- livorno niedrig
- practico 25 | 80
- practico 25 | 80 plus
- floorline 15 | 80

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Dear valued customer,

With your decision to purchase a side rail from Hermann Bock GmbH, you will receive a durable care product with superior functionality at the highest safety level.

This product was designed with a focus on persons in need of care, whose confidence must be reinforced and whose life needs protection. With this health care product, we meet these requirements.

We urge you to prevent potential malfunctions and the risk of accidents by complying strictly with the safety and operating instructions and by carrying out the necessary maintenance.

A handwritten signature in black ink that reads "Klaus Bock". The signature is written in a cursive, flowing style.

Klaus Bock

## Table of contents

<b>1</b>	<b>Preface and general instructions .....</b>	<b>4</b>
1.1	Reference to the assembly and operation manual for the nursing care bed ...	4
1.2	Definition of person groups .....	4
1.3	Safety instructions.....	4
1.4	Service life / warranty .....	5
<b>2</b>	<b>General description of the functions.....</b>	<b>6</b>
2.1	Construction design and function .....	6
2.2	Mounting variants - homecare and- residential care beds .....	6
2.3	Removal of the existing side rails .....	7
2.4	Assembly of the side rail SR2 on homecare beds.....	8
2.5	Assembly of the side rail SR2 on residential care beds .....	10
2.6	Disassembly of the side rail SR2 .....	11
2.7	Operation of the side rail SR2 .....	11
2.8	Transport, storage and operating conditions.....	12
2.9	Disposal .....	13
<b>3</b>	<b>Cleaning, maintenance and disinfection .....</b>	<b>13</b>
<b>4</b>	<b>Regular inspections with service.....</b>	<b>13</b>

## 1 Preface and general instructions

This assembly and operation manual describes the assembly of the side rails system SR2 - for the defined nursing care beds of Hermann Bock GmbH, and use of the side rail system SR2 . Read these assembly instructions completely to avoid damage due to incorrect assembly and handling.

Assembly of the side rail system is only to be carried out by qualified personnel and must under no circumstances be carried out by a private user.

**The installation of the side rail system SR2 is approved for the defined care beds from the production date January 2016. Preparations on the nursing care bed are specified in supplementary instructions.**

**This assembly and operation manual contains safety instructions. All persons who work with the nursing care beds must be familiar with the contents of these instructions. Improper operation can result in personal injuries.**

### 1.1 Reference to the assembly and operation manual for the nursing care bed

The assembly and operation manual of the respective Hermann Bock GmbH nursing care bed must be observed.

#### **Bock Note**

If you have received these instructions for assembly and use, then your product has additional equipment that is not described in the main instructions for assembly and use. Therefore, please attach these instructions to the main assembly and operating instructions.

### 1.2 Definition of person groups

#### *Qualified personnel*

Employees of the operator are referred to as qualified personnel. They are entitled to deliver the nursing care bed, assemble, dismantle and transport it, based on their training or instructions. Besides knowing how to operate, mount and disassemble the nursing care bed, these persons must be instructed according to the guidelines concerning the cleaning and disinfection of the nursing care bed.

### 1.3 Safety instructions

The intended use/operation of all moving parts is as important for the safety of the persons in need of care as well as for the relatives and the caregivers/nursing staff to avoid

potentially dangerous situations. The prerequisite for this is the correct assembly and operation of the side rails. When operating the side rails, it is also essential to take the individual physical constitution of the person in need of care into account, as well as the type and extent of the disability.

These recommendations apply particularly:

- if the person in need of care or the caregivers could be at risk due to those accidental adjustments;
- if the side rails are in a raised position and there could be danger of trapping and crushing,
- if children are unsupervised in the room with the bed.

The side rails should only be operated by trained caregivers or relatives or in the presence of trained persons.

When adjusting the side rails, particular care must be taken to ensure that no limbs are located in the side rails within the adjustment range.

Please note that the safety instructions in the main assembly and operating instructions for the nursing care bed also applies.

When using the side rails on incompatible beds, entrapment may occur.

#### **1.4 Service life / warranty**

This side rail system SR2 has been designed, engineered and manufactured to function safely over a long period of time. The service life depends on operating conditions and frequencies. Please refer to the main assembly and operating instructions for your nursing care bed.

**Attention: Unauthorised technical changes to the product voids all warranty claims.**

**This product is not approved for the North American market, particularly not for the United States of America (USA). Distribution and use of the nursing care bed in these markets, including through third parties, is prohibited by the manufacturer.**

## 2 General description of the functions


### 2.1 Construction design and function

#### *The side rail system*


All side rail variants serve to protect against falling out of bed. The side rail system SR2 allows a safe and convenient entry and exit at the same time. By means of a simple release mechanism, each individual side rail element can be brought into a raised or lowered position.

### 2.2 Mounting variants - homecare and- residential care beds

Bracket for homecare beds

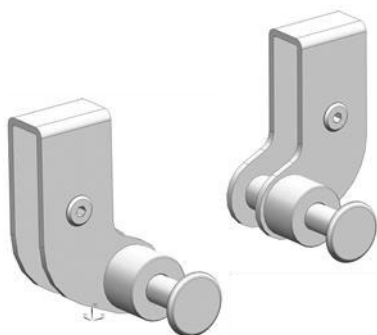Bracket for residential care beds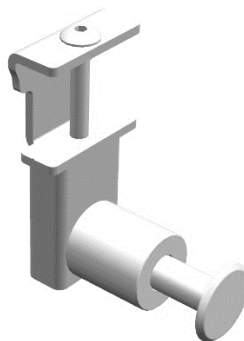

Suitable for the following nursing care beds:

- domiflex 1
- domiflex low
- domiflex classic low
- domiflex 2
- domiflex 2 wash
- domiflex 3
- domiflex 3 | 185
- combiflex
- belluno
- practico economic

Suitable for the following nursing care beds:


- livorno low
- practico 25|80
- practico 25|80 plus
- floorline 15|80

### 2.3 Removal of the existing side rails


When removing the existing rails, you must follow the steps for assembly of the side rails given in the main assembly and use instructions of the nursing care bed in reverse order. The further disassembly of the fittings is explained here.

Move the nursing bed to the highest position for easier dismantling.


The two bars must be removed.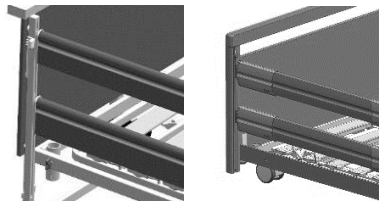


The fitting must also be removed from the C-guide rail.

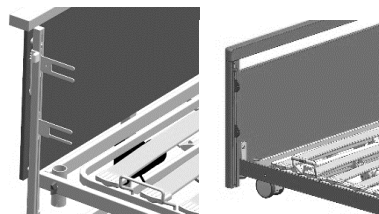Loosen the screw at the bottom of the C-guide rail, remove the fitting downwards and put the screw connection back into place.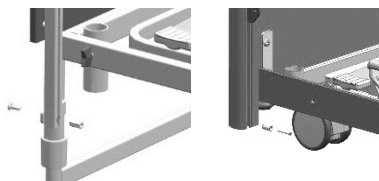


Now the C-rail is without fittings and prepared for the further installation of the hinged side safety catch.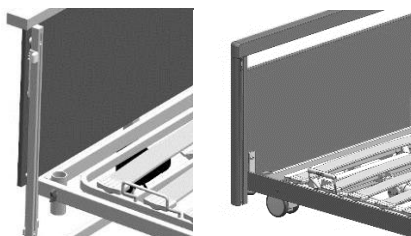


## 2.4 Assembly of the side rail SR2 on homecare beds

Move the nursing care bed to the highest position for easier assembly.


Place the two brackets for the homecare beds with the larger hole facing outwards on the longitudinal frame towards the respective end. The curvatures must point towards each other.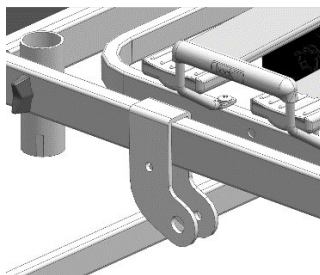


Connect the bracket with the screw M8x40mm and the lock nut, screw it only loosely.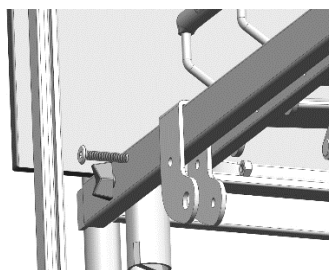

The joint must still be able to slide on the longitudinal frame.

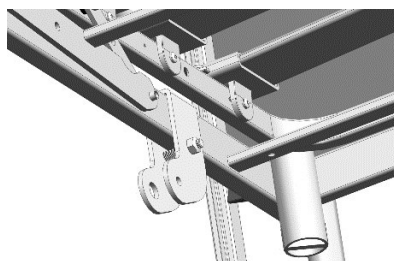
Insert the shoulder bolt into the side rail.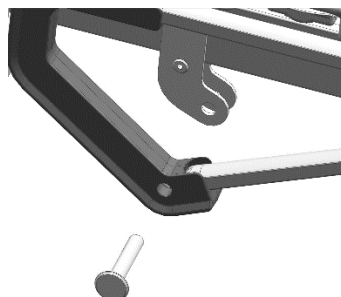


Slide the spacer onto the shoulder bolt.

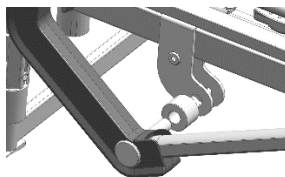
Push the side rail with the attachment bolt into the larger hole as far as it will go.

Between the U-profile, adjust the sleeve as a spacer when inserting.

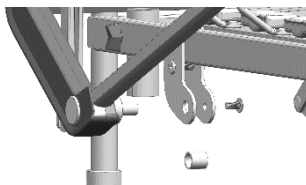
From the inner side, tighten the screw as shown.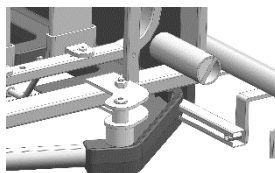


Move the side rail SR2 to the upper position.

**Attention: The dimension between the vertical outer edge of the side rail and the nearest possible component must be 30 mm in the horizontal direction.**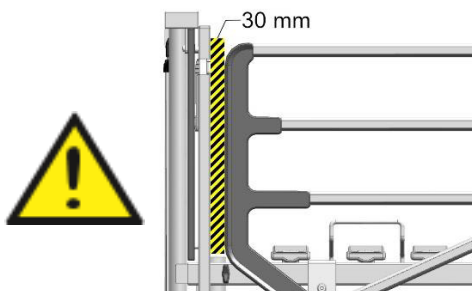

Tighten the upper screw M8x40mm.


Functional check. Lower the side rails and put them back in place.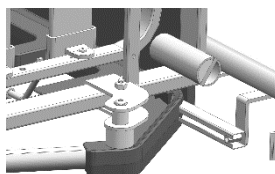


## 2.5 Assembly of the side rail SR2 on residential care beds

Move the nursing care bed to the highest position for easier assembly.


Holes are provided on the longitudinal frame of the nursing care bed for the side rails. These are sealed with plastic plugs, please remove these plastic plugs.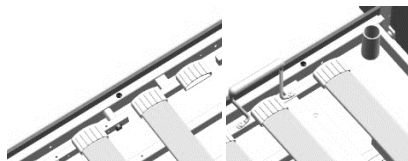

The brackets are now mounted on the longitudinal frame. Note that the lower, larger hole faces outward.

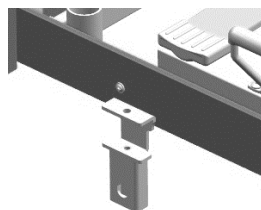
To do this, take the bracket as shown and fix it by inserting the M8x60mm screw through the component and the frame and tighten it.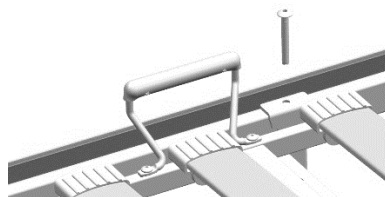

The attachment bolt is now pushed through the side rail SR2, with the attachment pointing outwards.

Then slide the spacer over the bolt from the inner side.

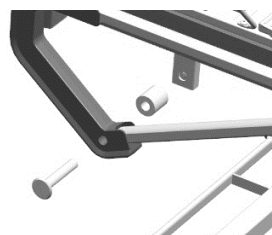
The side rail SR2 is now pushed into the lower, larger hole with the bolt as far as it will go.

Tighten the screw from the inside as shown in the picture.

Functional check. Lower the side rails and put them back in place.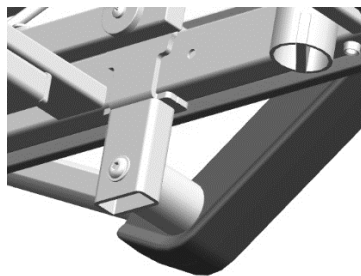


## 2.6 Disassembly of the side rail SR2

Disassembly of the side rail SR2 is done in reverse order to assembly.


## 2.7 Operation of the side rail SR2

Each side rail element can be adjusted independently. The release button for the adjustment is located in the middle of the side rail SR2, this release button must be pulled downwards and with the other hand pulled on the corresponding longitudinal bar of the side rail.


To lower a side rail SR2, grasp the upper bar of the side rail with one hand and release the release button with the other hand by pulling it slightly downwards.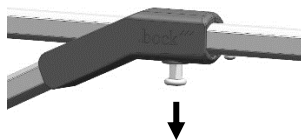


The side rails SR2 can then be lowered in one longitudinal movement

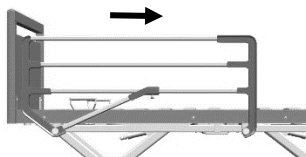The side rail SR2 is now in the lowered position.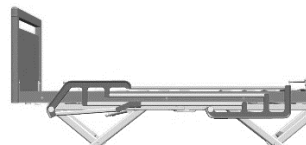

To move the side rail SR2 to the upper position, grasp the upper side rail with one hand and pull the side guard rail upwards into the upper position, the release button clicks audibly into place. The side rail SR2 is now in a pulled-up position.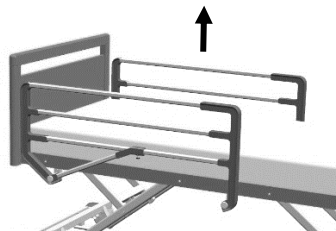

### Hazard note from Bock

Please note the increased risk of trapping caused by the raised side rails when adjusting the backrest and upper leg rest.

When using different mattress thicknesses, the minimum height must not be below 22 cm, measured from the upper edge of the side rails above the mattress, without compression.

Therefore, the following measurements must be observed for safe use of the side rails:

- Height above the mattress:  $\geq 220\text{mm}$
- Gap dimensions between bars and to the lying surface:  $<120\text{mm}$
- Gap dimensions between side rail and end piece:  $<60\text{mm}$  or  $>318\text{mm}$

## 2.8 Transport, storage and operating conditions

	Transport and storage	Operation
<b>Temperature</b>	0°C to +40°C	10°C to +40°C
<b>Relative humidity</b>	20% to 80%	20% to 70%
<b>Air pressure</b>	800hPa to 1060hPa	

## **2.9 Disposal**

The individual material components made of plastic and metal are recyclable and can be recycled in accordance with legal requirements. In general, when disposing of the device, make sure that the components to be disposed of are not infectious or contaminated.

## **3 Cleaning, maintenance and disinfection**

By following the care instructions in the main assembly and use instructions, the usability and visual condition of your nursing care bed and side rails will be maintained for a long time. The chapter "Cleaning, care and disinfection" for your nursing care bed applies to the side rails.

## **4 Regular inspections with service**

Regular inspections facilitate the maintaining of the highest possible safety level, and are considered to be an important safety precaution. Medical devices must be inspected regularly in terms of safety according to the stipulated regulations of the manufacturer and the generally accepted rules of technology. The chapter "Regular inspections with service" of your nursing care beds applies to the side rails.

Hermann Bock GmbH  
Nickelstr 12  
D-33415 Verl, Germany

Tel.: +49 52 46 92 05 - 0  
Fax: +49 52 46 92 05 - 25  
Internet: [www.bock.net](http://www.bock.net)  
E-Mail: [info@bock.net](mailto:info@bock.net)

#### Our SALES PARTNERS

Our business partners pursue the same strategy as we do: quality, innovation and above-average standards that are internationally recognized. You can rely on our business partners as you can rely on us.

Please note that only our authorised personnel and our sales partners can provide training, supply of spare parts, repairs, inspections and other service. Otherwise, all warranty claims will be void.

A listing of our current distributors can be found under [www.bock.net/kontakt/vertriebspartner](http://www.bock.net/kontakt/vertriebspartner)

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A handwritten signature in black ink, reading "Klaus Bock". The signature is written in a cursive, flowing style with a large initial 'K'.

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©